🕀 DiaMedical USA



Assembly Instructions

for Headwall Floor Stand

Hardware List (Left to Right)



- (A) Bolt 5/16" x 3"
- (B) Bolt 3/8" x 1 ¼"
- (C) Bolt 3/8" x 1"

Part List (Left to Right)



- (D) Flat Washer 3/8"
- (E) Flat Washer 5/16"
- (F) Lock Washer 3/8"



- (G) Nylon Lock Nut 3/8"
- (H) Hex Nut 3/8"
- (J) Nylon Lock Nut 5/16"

Tools Required (Not Provided)

- 9/16" wrench
- ½" wrench



- (K) BT-FLSTAND-TUBE
- (L) BT-FLSTAND-CROSSBAR
- (M) BT-FLSTAND-BASE

🕀 DiaMedical USA



INSTALLATION INSTRUCTIONS

1. Locate position of the installation hole on BT-FLSTAND-TUBE and proceed to install to BT-FLSTAND-BASE as follows using the following hardware: 1 of (B), 2 of (D) and 1 of (G)



- 2. Repeat above step for the second BT-FLSTAND-TUBE also using 1 of (B), 2 of (D) and 1 of (G)
- 3. Locate and position the BT-FLSTAND-CROSSBAR across as is shown below using 2 of (C), 4 of (D) and 2 of (G):



 Repeat above step by setting the BT-FL-STAND-CROSSBAR diagonal to the BTFL-STAND-CROSSBAR inserted in the above step. This is shown below:




5. Insert the BT-FLSTAND-CROSSBAR across the two BT-FLSTAND-TUBE using 2 of (A), 4 of (E) and 2 of (J). This is shown below:



- 6. Tighten all nuts and bolts to ensure rigidity of the floor stand.
 - 7. Installation is now complete.



DiaMedical USA | 7013 Orchard Lake Rd, Suite #110 West Bloomfield, MI 48322 P: (877) 593-6011 | F: (248) 671-1550 | www.DiaMedicalUSA.com

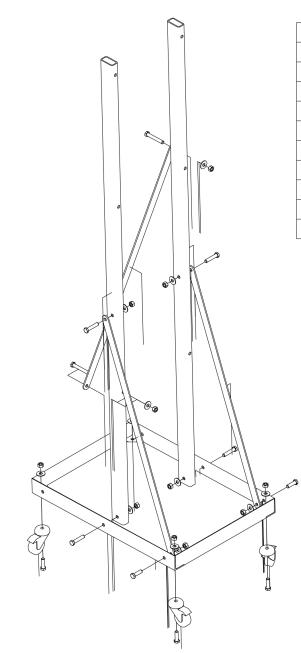

PARTS LIST			
ITEM	QTY	PART NUMBER	DESCRIPTION
3	1	BT-FLSTAND-BASE	Base Frame
4	2	BT-FLSTAND-TUBE	Rectangular Tube
5	3	BT-FLSTAND-CROSSBAR	Cross bar support
6	4	M-CAS-FLOOR1	Casters
7	4	H-X-CS-0498-17	Screw
12	6	H-X-CS-0498-12	Screw
14	2	H-X-CS-0498-25	Screw
17	12	H-X-NLN-0498	Nut
18	12	H-X-WASHER-FH-04	Washer



DiaMedical USA | 7013 Orchard Lake Rd, Suite #110 West Bloomfield, MI 48322 P: (877) 593-6011 | F: (248) 671-1550 | www.DiaMedicalUSA.com